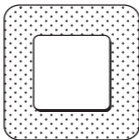
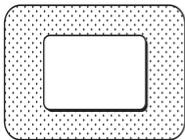
# Mölnlycke<sup>®</sup> Aldress<sup>®</sup>

Single use  
Usage unique

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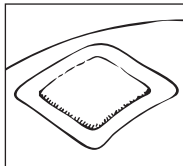
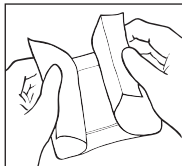
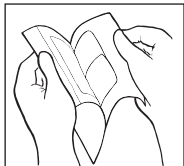
**Product description**

Alldress is a sterile, self-adhesive and absorbent dressing.

**Alldress consists of:**

- an absorbent wound pad with an integrated non-adherent wound contact layer.
- an adhesive border with water based acrylic adhesive. The adhesive is skin friendly and free from organic solvents. The adhesive area is covered with two overlapping release liners enabling aseptic application.
- an outer film which is breathable but waterproof, providing a barrier to external contaminants.

Dressing material content: Acrylic adhesive, polyethylene net, viscose nonwoven, polyester nonwoven, polyurethane film.

**Indications for use**

Alldress is intended to absorb, protect and maintain a moist environment in open and closed wounds. Alldress is intended for both dry and exuding wounds.

**Precautions**

- Do not use Alldress on patient with known hypersensitivity to acrylic adhesives or other ingoing materials/components of the product.
- Do not use on fragile skin. If used on fragile skin it may cause skin irritation.
- Do not stretch the dressing when applying.
- If you see signs of infection e.g. fever or the wound or surrounding skin becoming red, warm or swollen, consult a health care professional for appropriate treatment.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if inner package (sterile barrier) is damaged or opened prior to use. Do not re-sterilise.

**Instruction for use**

Alldress can be used by lay persons under supervision of health care professionals.

1. Cleanse the wound in accordance to clinical practice. Dry the surrounding skin thoroughly. Usage of moisturizers, gels or ointments on the skin surrounding the wound may affect the adhesion and should be avoided.
2. Choose a product size so that the wound pad well exceeds the size of the wound.
3. Remove the release liners and apply the sticky side to the skin without stretching the dressing. Ensure that the absorbent pad covers the wound.
4. Gently press the border to ensure proper adhesion to the skin.
5. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

Care should be taken to ensure that Alldress is not applied under tension, to prevent shear forces causing damage to the skin. When applying over joints be careful to apply in such a way that it does not impair flexibility of the joint. After some surgical interventions excessive swelling may occur. In such case it could be better to use a more flexible dressing in order to minimise skin tension.

Alldress can be used both as a primary dressing or as a secondary dressing together with for example gels and alginates.

Alldress should be changed before the dressing is fully saturated, at signs of leakage or as indicated by clinical practice.

Disposal should be handled according to local environmental procedures.

**Other information**

If any serious incident has occurred in relation to the use of Alldress, it should be reported to Mölnlycke Health Care.

Alldress® is registered trademark of Mölnlycke Health Care AB.

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Alldress® er skráð vörumerki Mölnlycke Health Care AB.