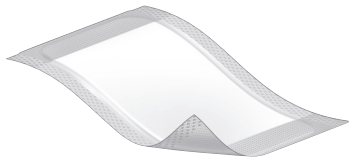
# Mölnlycke®

# Mesorb®

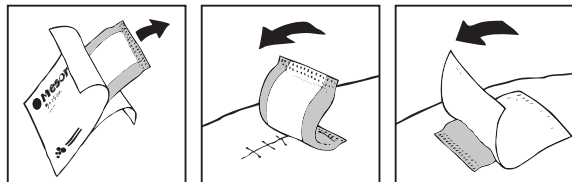
Manufacturer

 **Mölnlycke Health Care AB**  
Gamlestadvägen 3C, Box 13080  
SE-402 52 Göteborg, Sweden

[www.molnlycke.com](http://www.molnlycke.com)

## Mesorb®

[www.molnlycke.com/symbols](http://www.molnlycke.com/symbols)

Single use  
Usage unique

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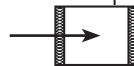
**Product description**

Mesorb is a highly absorbent cellulose pulp dressing with a core of cellulose pulp, fluid-repellent nonwoven backing and a polypropylene nonwoven wound contact layer.

Mesorb is designed to absorb and retain large quantities of wound exudate within the cellulose pulp core. The nonwoven fluid-repellent backing prevents external contamination of the wound and protects clothes and bed linen from exudate strike-through. The polypropylene nonwoven wound contact layer will not absorb wound exudate, reducing adherence to the wound surface.

Dressing material content: Polypropylene (wound contact layer, backing, hotmelt adhesive base), Cellulose.

**Indications for use**

Mesorb is intended for use in moderate to heavily exuding wounds.

Mesorb can be used as a primary dressing (directly on the wound), or as a secondary dressing where high absorbency is required.

**Precaution(s)**

- Do not use Mesorb on patient and/or user with known hypersensitivity to the materials/components of the product.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.
- Do not use after expiry date. If the product is used after the expiry date product properties cannot be ensured.

**Instructions for use**

Mesorb can be used by lay persons under supervision of health care professionals.

Application:

1. Clean the wound carefully if needed, and dry the surrounding skin.
2. Apply Mesorb directly on the wound area, or as a secondary dressing over a dressing that transfers exudate, with the white side of Mesorb facing the wound and the blue back side facing up. Secure at least 1 cm overlap on the surrounding skin.
3. Fix in place with a suitable fixation device.
4. Mesorb can be used under compression bandages.

Frequency of change:

Mesorb should be changed according to the clinical condition of the wound, or before saturated. If the condition of the wound deteriorates, discontinue use and consult a healthcare professional.

Maximum wear time of each dressing is 7 days.

**Special storage conditions and handling conditions**

Disposal should be handled according to local environmental procedures.

**Other information**

If any serious incident has occurred in relation to the use of Mesorb, it should be reported to Mölnlycke Health Care.

Mesorb is a registered trademark of Mölnlycke Health Care AB.

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